

Test 28: Film viewer condition

Perceived brightness, colours and must be clean and uniformly illuminated 6 monthly

SITE: _____					
TEST 28: FILM VIEWING CONDITIONS					
DATE	NAME	RESULTS	PASS	FAIL	CORRECTIVE ACTION

PURPOSE

For correct interpretation of diagnostic images.

To ensure that view boxes are clean and light levels are kept consistent throughout.

PROCEDURE:

Visually check viewing boxes for similar brightness and colour.

Viewing boxes must be clean and uniformly illuminated.

If not, clean surface of the view box.

If a bulb or tube fails, it is best to replace all.

