

Test 2

Gonad shields, lead rubber aprons and gloves

3 monthly

SITE: _____				
TEST NO. 2: GONAD SHIELDS AND LEAD APRONS VISUAL CHECK - RM _____				
DATE	NAME	NO. OF APRONS PASSED	NO. OF APRONS FAILED	CORRECTIVE ACTION

EQUIPMENT REQUIRED

Screening x-ray unit (if not available, use a general purpose unit and 35 cm x 43 cm cassettes).

METHOD

Spread article to be tested on the table top.

Use screening system to examine the whole article.

Use different kV's.

If a general purpose x-ray unit is to be used, place cassettes under test article and make separate exposures. Ensure that the whole article is examined.

“Screen for cracks”

