



DEPARTMENT OF HEALTH
DIRECTORATE: RADIATION CONTROL



GUIDELINES

BONE DENSITOMETER – SHIELDING, MONITORING AND POSITIONING OF OPERATORS

(The following recommendations serve as guidelines only – final requirements can only be made after an inspection by an official of this Directorate.)

1. Shielding requirements¹

- 1.1 Walls – dry walls should provide sufficient shielding with a workload of less than 7500 examinations per year (<30 examinations / day)
- 1.2 Protective screen (preferably transparent lead-acrylic shielding).
Only required if the distances below are not kept by the operator or the workload is more than 30 examinations / day.

2. Operator: Positioning

- 2.1 Pencil beam densitometers:
 - The operator shall be positioned at least 1 metre from the edge of the scan table.
- 2.2 Fan beam densitometers:
 - The operator shall be positioned at least 2 metres from the edge of the scan table.

3. Personal Monitoring

Personnel are required to wear personal dosimeters if the distances above are not kept and a protective screen not used.

Ref: CRCPD Publication E-06-05 (October 2006) – Technical White Paper: Bone Densitometry

----- ooOOoo -----